



Collection Procedure

1. Remove swab from plastic packaging. Avoid touching swab applicator below the pink molded breakpoint as this could lead to contamination and incorrect test results.
2. Pass Stool into a clean, dry pan.
3. Collect a small amount of stool by inserting all the tip of the swab stool sample and rotate it.
4. Remove and examine to make sure fecal material is visible on the swab.
5. Once completed, insert swab back into its media vial tuube, brush end first.
6. Holding the swab shaft close to the rim of the tube, and keeping the tube away from your face, break the applicator shaft at the pink breakpoint indication line.
7. Screw the cap on tightly to prevent leakage.

Transport Procedure

1. Confirm patient information on requisition form matches the information on collection tube.
2. Place the collection tube into specimen bag along with ordering requisition form, seal specimen bag.
3. The package is not ready for bulk shipment via UPS/FedEx or Courier.



For additions questions, contact VeroDx Customer Success

✉ accounts@verodx.com

☎ (919) 341-1256