



Transport Procedure

Male

1. Wash Hands using soap and warm water.
2. If uncircumcised, retract foreskin and wipe end of penis with a fresh towelette.
3. Allow a small amount of urine to fall in toilet bowl. **DO NOT** touch inside of sterile urine cup.
4. Collect urine stream into sterile urine cup. Be sure 3/4 of cup is filled.

Female

1. Wash Hands with soap and warm water.
2. Spread the labia (folds of skin) apart with one hand and wipe with a towelette. Wipe from front to back.
3. Continue holding the labia apart. Allow a small amount of urine to fall in toilet bowl. **DO NOT** touch inside of sterile urine cup.
4. When using a potty collection pan, urinate in pan and transfer urine to sterile urine cup. Be sure 3/4 of cup is filled.

Collection Procedure

1. Urinate into the sterile cup and close the lid.
2. Peel the Yellow sticker off the cup (Please do NOT touch the needle as it is very sharp).
3. Insert the test tube provided upside down into the needle filling the tube until full.
4. Put the barcode from the requisition on the tube.
5. Put the tube in a biohazard bag provided.
6. Place the biohazard bag in the supplied UPS or FedEx Bag and ship to the lab.



For additions questions, contact VeroDx Customer Success

✉ accounts@verodx.com

☎ (919) 341-1256