



## Collection Procedure

1. Remove swab from plastic packaging. Avoid touching the swab applicator below the pink molded breakpoint.
2. Carefully insert swab into the opening of vagina, about two inches, and gently rotate the swab for 10 to 30 seconds. Make sure swab touches the walls of the vagina so that moisture is absorbed into the swab.
3. Withdraw swab without touching skin.
4. Insert swab back into media vial, brush end first.
5. Holding the swab shaft close to the rim of the tube, and keeping the tube away from your face, break the applicator shaft at the pink breakpoint line.
6. Screw the cap on tightly to prevent leakage.

## Post Collection/Transport Procedure

1. Confirm patient information on ordering requisition form matches patient information on specimen tube.
2. Place sample into specimen bag, along with the ordering requisition form and seal specimen bag.
3. The package is now ready for bulk shipment via UPS/FedEx or courier.



For additions questions, contact VeroDx Customer Success

✉ [accounts@verodx.com](mailto:accounts@verodx.com)

☎ (919) 341-1256