



## Collection Procedure

1. Wash the wound with saline and wipe lightly.
2. Remove the swab from its plastic packaging. Avoid touching the swab applicator below the pink molded breakpoint.
3. Consult with physician about wound collection. Generally, roll the swab in the wound by applying appropriate pressure. Try to get enough material on swab.
4. Insert the swab back into its media vial, brush end first.
5. Holding the swab shaft close to the rim of the tube, and keeping the tube away from your face, break the applicator shaft at the pink breakpoint line.
6. Screw the cap on tightly to prevent leakage.

## Post Collection/Transport Procedure

1. Confirm patient information on the ordering requisition matches patient information on specimen tube.
2. Place the specimen tube into the specimen bag, along with the ordering requisition form and seal.
3. The package is now ready for bulk shipment via UPS/FedEx or courier.



For additions questions, contact VeroDx Customer Success

✉ [accounts@verodx.com](mailto:accounts@verodx.com)

☎ (919) 341-1256